Compressor Nebuliser

- 1) Piston compressor
- 2) Power lead
- 3) ON/OFF Switch
- 4) Air filter compartment
- 5) Nebuliser
 - a- Vapouriser head
- 6) Air hose
- 7) Mouthpiece
- 8) Adult face mask
- 9) Child face mask
- 10) Replacer air filter
- 11) Assembling nebuliser kit
- 12) Nose piece
- 13) Air filter

Dear Customer,
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. This device has been specifically designed to make treatment more pleasant for children, whilst ensuring all the quality and safety standards of Microlife products. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the name of the nearest Microlife distributor. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products.
Stay healthy - Microlife AG!

1. Explanation of symbols

⚠ Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

📖 Read the instructions carefully before using this device.

👤 Type BF applied part

📦 Class II equipment

SN Serial number

REF Reference number

🏭 Manufacturer

🔘 ON

🔘 OFF

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water.

CE0123 CE Marking of Conformity

2. Important Safety Instructions

Follow instructions for use. This document provides important production operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
Install instructions in a safe place for future reference. Do not operate the unit in presence of any anesthetic mask inflammable with oxygen or nitrogen protoxide. This device is not suitable for anaesthesia and lung ventilation.
This device should only be used with original accessories as shown in these instructions.
Do not use this device if you think it is damaged or notice anything unusual.
Never open this device.
This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the Technical Specifications section.

Water and droplets

contamination and dust

direct splash

heat and cold

Comply with safety regulations concerning the electrical devices and in particular:

Never touch the device with wet or moist hands. Place the device on a stable and horizontal surface during its operation.

Do not plug the power cord or the device itself to sockets if the device is not switched off.

The power plug is a safety element from the grid power; keep the plug accessible when the device is in use.

Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating of the power supply network. Do not use the device if the power supply network does not fit your wall socket; contact qualified personnel for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and of extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention to the extension cables and to the safety of the cables, including on adapters and extension cables.

Do not let the unit plugged in when not in use; unplug the device from the wall socket when it is not in operation.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to the device and to the objects, for which the manufacturer cannot be held responsible.

Do not replace the power lead of this device. In case of power lead damage, contact a technical service center authorized by the manufacturer for its replacement.

The power supply cord should always be fully retracted in order to prevent dangerous overheating.

Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed by your or your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.

Check in the medication instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same accessories for more than one person.

Never bring the nebuliser over 60° C.

Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations.

Do not use the device at a distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised, some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

This device is not a toy! Keep your child during treatment.

Use of this device is not intended as a substitute for a consultation with your doctor.

The technical specifications may change without prior notice.

3. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

Assemble the nebuliser kit (1). Ensure that all parts are complete.

Fill the nebuliser with the inhalation solution as per your doctor's instructions to ensure that you do not exceed the maximum level.

Connect the nebuliser with the air hose (6) to the compressor (1) and the power lead (2) into the socket (230V 50 Hz AC).

To start the treatment, set ON/OFF switch (3) into the «ON» position.

The mouthpiece gives you a better drug delivery to the lungs.

Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.

Use all accessories including the nose piece (2) as prescribed by your doctor.

During inhalation, sit upright and relaxed at a table and not in an armchair, in order to avoid compressing your respiratory airways and impairing the treatment effectiveness. Do not lie down while inhaling. Stop inhalation if you feel unwell.

After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «OFF» to turn the device and unplug it from the socket.

Empty the remaining medication from the nebuliser, and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On 10 min. Off. Switch the power lead after 30 min. use and wait for another 30 min. before you resume treatment.

The device requires no treatment.

No modification to the device is permitted.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and prevent infections after each treatment. Use only soft and dry cloth with non-abrasive cleaners to clean the compressor.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Cleaning and disinfecting of the accessories

Follow carefully the cleaning and disinfecting instructions of the accessories to ensure the safety and the performance of the device and success of the therapy.

Before and after each treatment

Disassemble the nebuliser (5) by turning the top counter-clockwise and remove the nebuliser from the device. Wash the components of the disassembled nebuliser, the mouthpiece (7) and the nose piece (2) by using tap water; do not boil water for 3 minutes. Reassemble the nebuliser (5) and the nose piece (2) by using tap water; switch the device on and let it work for 10-15 minutes.

Wash masks and air tube with warm water.

Only use cold disinfecting liquids following the manufacturer's instructions.

Do not boil nor autoclave the air tube and masks.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife Service (see last period).

Replacement of the nebuliser

Replace the nebuliser (5) after a long period of inactivity, in cases where it shows deformities, breakage, or when the device does not work. Please read the instructions. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters!

In normal conditions of use, the air filter (13) must be replaced approximately after 500 working hours or after every 10-12 treatments. If the filter shows a grey or brown colour or is wet, replace it. Extract the filter and replace it with a new one.

Do not try to clean the filter for reusing it.

Use only the original filter as serviced or maintained while in use with a patient.

Only use original filters! Do not use the device without filter!

6. Malfunctions and Actions to take

The device cannot be switched on

Ensure the power lead (2) is correctly plugged into the socket.

Ensure the ON/OFF switch (3) is in the position «ON».

Make sure that the device has been operating with the correct voltage indicated in this manual (30 min On / 30 min Off).

The nebuliser functions poorly or not at all

Ensure the air hose (6) is correctly connected at both ends.

Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.

Ensure the nebuliser (5) is fully assembled and the operating lid (1) is closed correctly and not obstructed.

Ensure the required medication has been added.

7. Guarantee

This device is covered by a 5 year guarantee from the bottom of the unit. Corresponds to the mains rating of the power supply network. Do not use the device if the power supply network does not fit your wall socket; contact qualified personnel for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and of extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention to the extension cables and to the safety of the cables, including on adapters and extension cables.

Do not let the unit plugged in when not in use; unplug the device from the wall socket when it is not in operation.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to the device and to the objects, for which the manufacturer cannot be held responsible.

Do not replace the power lead of this device. In case of power lead damage, contact a technical service center authorized by the manufacturer for its replacement.

The power supply cord should always be fully retracted in order to prevent dangerous overheating.

Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed by your or your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.

Check in the medication instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same accessories for more than one person.

Never bring the nebuliser over 60° C.

Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations.

Do not use the device at a distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised, some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

This device is not a toy! Keep your child during treatment.

Use of this device is not intended as a substitute for a consultation with your doctor.

The technical specifications may change without prior notice.

3. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

Assemble the nebuliser kit (1). Ensure that all parts are complete.

Fill the nebuliser with the inhalation solution as per your doctor's instructions to ensure that you do not exceed the maximum level.

Connect the nebuliser with the air hose (6) to the compressor (1) and the power lead (2) into the socket (230V 50 Hz AC).

To start the treatment, set ON/OFF switch (3) into the «ON» position.

The mouthpiece gives you a better drug delivery to the lungs.

Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.

Use all accessories including the nose piece (2) as prescribed by your doctor.

During inhalation, sit upright and relaxed at a table and not in an armchair, in order to avoid compressing your respiratory airways and impairing the treatment effectiveness. Do not lie down while inhaling. Stop inhalation if you feel unwell.

After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «OFF» to turn the device and unplug it from the socket.

Empty the remaining medication from the nebuliser, and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On 10 min. Off. Switch the power lead after 30 min. use and wait for another 30 min. before you resume treatment.

The device requires no treatment.

No modification to the device is permitted.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and prevent infections after each treatment. Use only soft and dry cloth with non-abrasive cleaners to clean the compressor.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Cleaning and disinfecting of the accessories

Follow carefully the cleaning and disinfecting instructions of the accessories to ensure the safety and the performance of the device and success of the therapy.

Before and after each treatment

Disassemble the nebuliser (5) by turning the top counter-clockwise and remove the nebuliser from the device. Wash the components of the disassembled nebuliser, the mouthpiece (7) and the nose piece (2) by using tap water; do not boil water for 3 minutes. Reassemble the nebuliser (5) and the nose piece (2) by using tap water; switch the device on and let it work for 10-15 minutes.

Wash masks and air tube with warm water.

Only use cold disinfecting liquids following the manufacturer's instructions.

Do not boil nor autoclave the air tube and masks.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife Service (see last period).

Replacement of the nebuliser

Replace the nebuliser (5) after a long period of inactivity, in cases where it shows deformities, breakage, or when the device does not work. Please read the instructions. We recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use original filters!

In normal conditions of use, the air filter (13) must be replaced approximately after 500 working hours or after every 10-12 treatments. If the filter shows a grey or brown colour or is wet, replace it. Extract the filter and replace it with a new one.

Do not try to clean the filter for reusing it.

Use only the original filter as serviced or maintained while in use with a patient.

Only use original filters! Do not use the device without filter!

6. Malfunctions and Actions to take

The device cannot be switched on

Ensure the power lead (2) is correctly plugged into the socket.

Ensure the ON/OFF switch (3) is in the position «ON».

Make sure that the device has been operating with the correct voltage indicated in this manual (30 min On / 30 min Off).

The nebuliser functions poorly or not at all

Ensure the air hose (6) is correctly connected at both ends.

Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.

Ensure the nebuliser (5) is fully assembled and the operating lid (1) is closed correctly and not obstructed.

Ensure the required medication has been added.

7. Guarantee

This device is covered by a 5 year guarantee from the bottom of the unit. Corresponds to the mains rating of the power supply network. Do not use the device if the power supply network does not fit your wall socket; contact qualified personnel for a replacement plug with that of a suitable one. In general, the use of adapters, simple or multiple, and of extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying attention to the extension cables and to the safety of the cables, including on adapters and extension cables.

Do not let the unit plugged in when not in use; unplug the device from the wall socket when it is not in operation.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to the device and to the objects, for which the manufacturer cannot be held responsible.

Do not replace the power lead of this device. In case of power lead damage, contact a technical service center authorized by the manufacturer for its replacement.

The power supply cord should always be fully retracted in order to prevent dangerous overheating.

Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed by your or your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.

Depending on the pathology, only use the treatment that is recommended by your doctor.

Only use the nose piece indicated by your doctor, paying special attention NEVER to introduce the bifurcations into the nose, but only positioning them as close as possible.

Check in the medication instruction leaflet for possible contraindications for use with common aerosol therapy systems.

Do not position the equipment so that it is difficult to operate the disconnection device.

For greater hygienic safety, do not use the same accessories for more than one person.

Never bring the nebuliser over 60° C.

Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations.

Do not use the device at a distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised, some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

This device is not a toy! Keep your child during treatment.

Use of this device is not intended as a substitute for a consultation with your doctor.

The technical specifications may change without prior notice.

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