microlife® GoldTemp Thermometer

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IMPORTANT SAFETY INSTRUCTIONS

EN

- Digital medical thermometer MT 1622 is intended to use for oral, axillar or rectal measuring of human temperature. Field of application: in medical institutions and at home for individual use. A display of values with an accuracy of 0.01 C. It can be recommended for family planning.
- The instrument may be used only for measuring body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60 °C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0.01 °C in 8 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

BATTERY REPLACEMENT

When the «▼»-symbole appears, battery is exhausted and needs replacing. Please remove the compartment lid of the thermometers and replace the exhausted battery by positioning the + at the top.

GUARANTEE

We grant you a 5 year guarantee after the date of purchase. Any damage caused by improper handling shall not be covered by the warranty. Battery and packaging are also excluded from the warranty. All other damage claims excluded. A warranty claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:
Date of purchase:

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Measuring methods / Normal body temperature

- In the armpit (axillary) / 34.7 - 37.3 °C**
Wipe the underarm with a dry towel. Place the measuring sensor ④ under the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait **at least 5 minutes**, regardless of the beep sound.
- In the mouth (oral) / 35.5 - 37.5 °C**
Do not eat or drink anything hot or cold 10 minutes before the measurement. The mouth should remain closed up to 2 minutes before starting a reading. Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor ④ must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air. If this is not possible due to blocked airways, another method for measuring should be used.
Approx. measuring time: 1 minute!
- In the anus (rectal) / 36.6 - 38.0 °C**
Attention: For the prevention of rectal perforation in children (younger than 3 years), we recommend using another measuring method, or the use of a thermometer with flexible tip. Carefully insert the measuring sensor ④ of the thermometer 2 to 3 cm into the anal aperture. The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.
Approx. measuring time: 1 minute!

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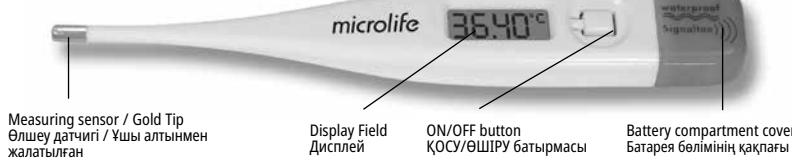
IMPORTANT SAFETY INSTRUCTIONS

Digital medical thermometer MT 1622 is intended to use for oral, axillary or rectal measuring of human temperature... The instrument may be used only for measuring body temperature!... The minimum measurement time until the signalling tone [beep] must be maintained without exception!

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON»... The thermometer is now ready for use.

USING THE THERMOMETER

Choose the preferred measuring method. During a measurement, the current temperature is displayed continuously... To prolong the life of the battery, turn it off after use by pressing the operating button briefly.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed... This value will disappear and the thermometer will switch to the normal measurement mode.

BATTERY REPLACEMENT

When the «M»-symbol appears, battery is exhausted and needs replacing... The thermometer is ready for use.

GUARANTEE

We grant you a 5 year guarantee after the date of purchase... The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

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